


We recently improved the faceplate controls for the Pro Fit. It is now easier to control the volume and change the memories by pressing a switch.

## User Switch Control Instructions

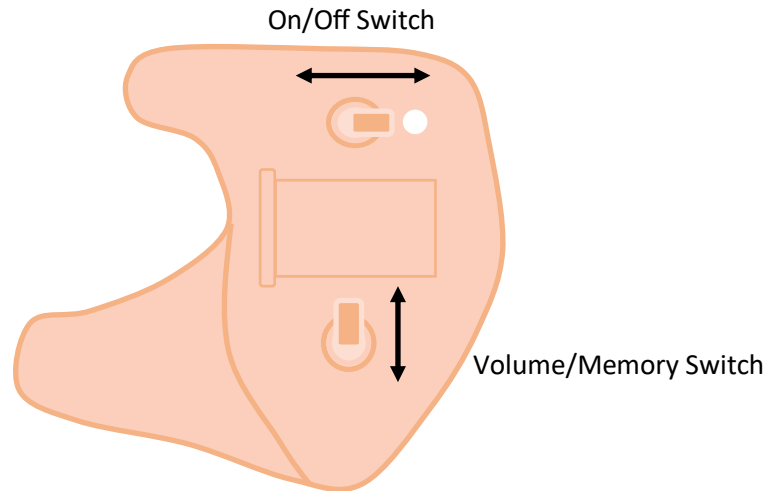

### **ON/OFF SWITCH:**

When switch is positioned towards the white dot, the device is off. When the switch is positioned opposite of the white dot (towards your face), the device is powered on.

**Rechargeable Devices** - The switch MUST be in the off position when charging. (towards white dot)

### **VOLUME/MEMORY SWITCH:**

#### **Changing Volume**

Push switch upward to increase volume.

Push switch downward to lower volume.

**\*Please Note** - ProFit Classic models will provide a single beep when the volume level reaches the maximum or minimum levels.

#### **Changing Memories**

Push and HOLD switch upward for 3 seconds to change memories upward. 1, 2, 3, 4

Push and HOLD switch downward for 3 seconds to change memories downward. 4, 3, 2, 1